# Blade<sup>XT</sup> Female Pyramid Adapter

## Instructions for Use

189527

EN	Instructions for Use	2
DA	Brugsanvisning	11
NO	Bruksanvisning	20
FI	Käyttöohjeet	29
SV	Bruksanvisning	38
RU	Инструкция протезиста	47
ZH	使用说明	56
AR	إرشادات الاستخدام	65

# Contents

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	4
3 Construction .....	5
4 Function.....	6
5 Maintenance .....	6
6 Limitations on Use .....	7
7 Fitting Advice.....	7
8 Technical Data .....	8
9 Ordering Information .....	9

---

# 1 Description and Intended Purpose

These Instructions for Use are intended for use by the practitioner and user unless otherwise stated.

The term *device* is used throughout these instructions for use to refer to the Female Pyramid Adapter.

Please read and ensure you understand all instructions for use, in particular all safety information and maintenance instructions.

## Application

This device is for use exclusively as part of a lower limb prosthesis.

The device connects a BladeXT to Blatchford modular limb components with a male pyramid receiver and allows for A-P shift, angular and rotational adjustment.

Intended for single user only.

For use with Blatchford modular limb components only.

## Features

- The A-P shift and rotation prior to tightening the pyramid screws and the alignment nut allows for alignment of the limb to suit the individual user.
- When tightened securely the device offers a secure way to connect two parts of the limb.

## Activity Level

This device is recommended for users that have the potential to achieve Activity Level 3 or 4. Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

### Activity Level 3

Has the ability or potential for ambulation with variable cadence.

Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

### Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

## Contraindications

There are no known contraindications if used in accordance with these instructions.

## Clinical Benefits

- Allows connection of the BladeXT to other Blatchford approved prosthetic componentry

## 2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Only combine this device with the Blatchford BladeXT device.

Any changes in the performance or function of the device unusual noises, excessive play or loss of alignment should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

Avoid exposure to extreme heat and/ or cold.

To reduce the risk of injury due to failure or loosening of the screw connections ensure the bolt threads are cleaned thoroughly before each installation.

The device is designed for prolonged submersion and suitable for immersion in fresh water only. Ensure any use of the device in water complies with the conditions given in *Limitations on Use*.

Be aware of finger trap hazard at all times.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified practitioner.

The user must not adjust or tamper with the setup of the device.

The user should be advised to contact their practitioner if their condition changes.

Always apply the specified torque values to the screws/ nuts. Never replace the screws with alternative screws. Always use the bolts specified.


Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

# 3 Construction

## Principal Parts

- Female Pyramid Titanium
- Bolt Titanium
- Pyramid Screws Stainless steel
- Alignment nut and washer Stainless steel

## Component Identification

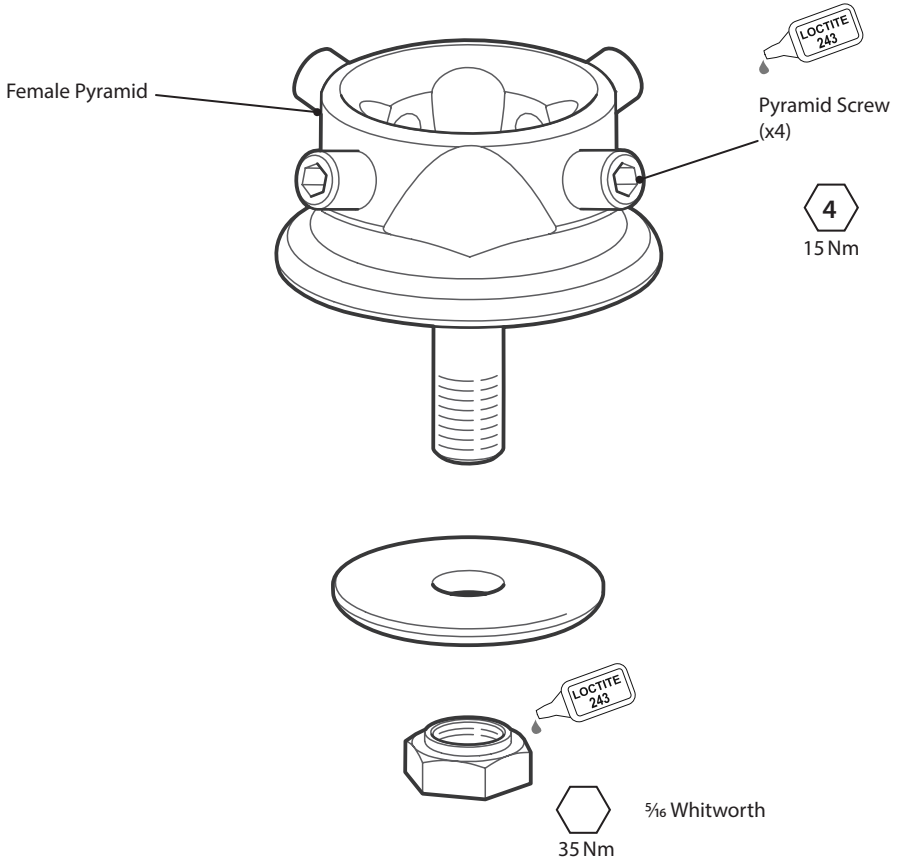---

## 4 Function

This device is designed to connect a BladeXT to a male pyramid receiver and allows for A-P shift angular and rotational adjustment.

The A-P shift and rotation prior to tightening the pyramid screws and the alignment nut allows for alignment of the limb to suit the individual user.

When tightened securely using Loctite and torque settings shown, the device offers a secure way to connect two parts of the limb.

---

## 5 Maintenance

Visually check the device regularly.

Report any changes in performance of this device to the practitioner/service provider.

Changes in performance may include:

- Instability
- Excessive play or loss alignment
- Any unusual noise

Inform the practitioner/service provider of any changes in body weight and/or activity level.

### **Cleaning**

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

***The remaining instructions in this section are for practitioner use only.***

Maintenance must be carried out only by competent personnel (practitioner or suitable trained technician).

Carry out the following routine maintenance at least annually:

- Ensure all screws are secure. If not remove and clean screws, reapply Loctite and tighten all screws to the correct torque settings.
- Check for visual defects that may affect proper function.

---

## 6 Limitations on Use

 **Only for use with BladeXT in conjunction with a Blatchford approved pyramid system.**

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

The product is waterproof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15 °C and 50 °C  
(5 °F to 122 °F).

Suitable for submersion

---

## 7 Fitting Advice

*The instructions in this section are for practitioner use only.*


Symptoms	Remedy
<ul style="list-style-type: none"><li>A recurring noise occurs between BladeXT and the adapter.</li></ul>	<ol style="list-style-type: none"><li>All bolts and nuts must be tightened to the torque ratings specified in this document.</li><li>Apply Loctite where applicable.</li></ol>
<ul style="list-style-type: none"><li>The adapter moves out of position.</li></ul>	<ol style="list-style-type: none"><li>User must not use the device until adjusted, repaired or replaced.</li></ol>

## 8 Technical Data

Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight ( <i>size 26</i> ):	130 g (4.6 oz)
Activity Level:	1–4
Maximum User Weight:	166 kg (275 lb)
Range of Adjustment:	±360° rotation ±7° angular
AP Shift: (Fitted to BladeXT)	35 mm
Build Height:	26 mm
Connections:	Proximal Female Pyramid Distal Single Bolt

---

### Build Height

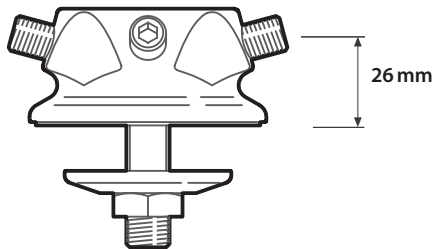---

## 9 Ordering Information

### Part Number

189527

Spare Parts	
M8x16 flat point socket set screw	910457
Bolt - flat align	180004
Alignment Washer	180003
Alignment Nut	180001

---

### Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

### CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

### Compatibility

Combination with Blatchford BladeXT device products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

### Warranty

This device is warranted for 24 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

## **Reporting of Serious Incidents**

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## **Environmental Aspects**

This product is constructed from recyclable materials. Where possible the components should be recycled in accordance with local waste handling regulations.

## **Retaining the Packaging Label**

The practitioner is advised to keep the packaging label as a record of the device supplied.

## **Trademark Acknowledgements**

BladeXT and Blatchford are registered trademarks of Blatchford Products Limited.

## **Manufacturer's Registered Address**

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**Blatchford Europe GmbH**
**Endolite India Ltd.**