Foot Shell Cover

Instructions for Use

539087, 539096

EN	Instructions for Use	2	SK	Návod na použitie	50
FR	Instructions d'utilisation	6	HU	Használati útmutató	54
DE	Gebrauchsanweisung	10	EL	Οδηγίες χρήσης	58
IT	Istruzioni per l'uso	14	LV	Lietošanas pamācība	62
ES	Instrucciones de uso	18	LT	Naudojimo instrukcija	66
NL	Gebruiksaanwijzing	22	ET	Kasutusjuhend	70
PL	Instrukcja użytkowania	26	DA	Bruksanvisning	74
PT	Instruções de utilização	30	NO	Bruksanvisning	78
CS	Návod k použití	34	FI	Käyttöohjeet	82
SL	Navodila za uporabo	38	SV	Bruksanvisning	86
BG	Инструкции за употреба	42	RU	Инструкция протезиста	90
HR	Upute za uporabu	46	ZH	使用说明	94
			AR	إرشادات الاستخدام	98

Blatchford:

Contents	2
1 Description and Intended Purpose	2
2 Safety Information	2
3 Construction	2
4 Maintenance	3
5 Limitations on Use	3
6 Assembly Instructions	3
7 Technical Data	4

1 Description and Intended Purpose

These instructions for use are intended for use by the practitioner and user.

The term *device* is used throughout these instructions for use to refer to Foot Shell Cover.

Please read and ensure you understand all instructions for use, in particular all safety information and maintenance instructions.

Application

This device is to be used exclusively as part of a lower limb prosthesis.

Intended for a single user.

The device is a cover designed to provide protection from the ingress of debris into the foot shell of Elite Blade, Javelin or EchelonVT products.

For information relating Activity Level, Contraindications and Clinical Benefits, please consult the associated Instructions for Use of Elite Blade, Javelin or EchelonVT as appropriate.

2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Report any changes in performance of this device to the practitioner/service provider.


3 Construction

Principal Parts


- Footshell Cover Neoprene

Component Identification

EchelonVT
Foot Shell Cover
(539096)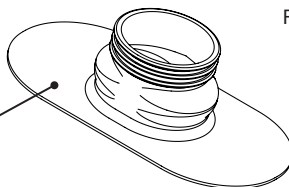

Elite Blade/Javelin
Foot Shell Cover
(539087)

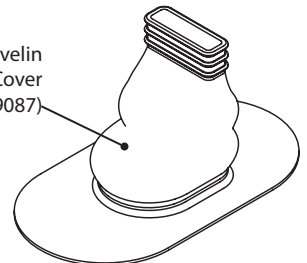4 Maintenance

Visually check the device regularly. Report any splits, tears or separation to the practitioner/service provider. The device has no serviceable parts and must be replaced if damaged.

If water or any other liquid enters the foot, it may be necessary to create a gap before inverting the foot and allowing it to dry.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

5 Limitations on Use

For information relating Limitation on Use please consult the associated Instructions for Use.

6 Assembly Instructions

The instructions in this section are for practitioner use only.

Use appropriate health and safety equipment at all times including extraction facilities.

For use if cover is to be adhered to the foot shell.

1

Roughen the top surface of the foot shell.

2

Fit the product into foot shell and apply Loctite 424 (926104) between the underside of the cover and top surface of the foot shell.

3

After pressing the two surfaces together until the adhesive has cured, trim excess material following the foot shell.

4


Grind edges flush with foot shell to finish.

7 Technical Data

Part No.	539087 (Elite Blade/Javelin)	539096 (EchelonVT)
Component Weight (prior to trimming)	47 g (1.5 oz)	36 g (1.25 oz)
Operating and Storage Temperature Range	-15 °C to 50 °C (5 °F to 122 °F)	-15 °C to 50 °C (5 °F to 122 °F)

Fitting Length

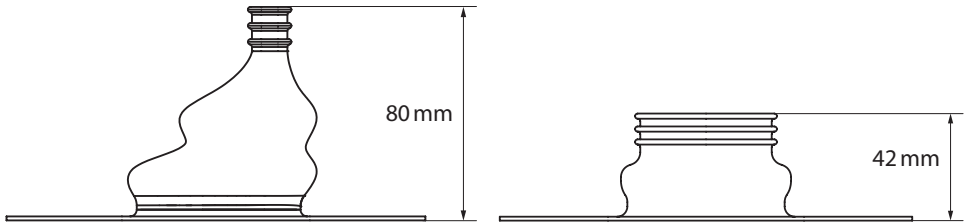Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical Device

Single Patient – multiple use

Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

Warranty

This device is warranted for 1 month.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Retaining the Packaging Label

The practitioner is advised to keep the packaging label as a record of the device supplied.

Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

Manufacturer's Registered Address

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Blatchford Europe GmbH
Endolite India Ltd.