ElanIC

User Guide

ELANIC22L1S - ELANIC30R8S
ELANIC22L1SD - ELANIC30R8SD

| | | |
|----|------------------------|----|
| EN | User Guide | 2 |
| FR | Guide de l'utilisateur | 15 |
| DE | Bedienungsanleitung | 28 |
| IT | Guida per il paziente | 41 |
| ES | Guía del usuario | 54 |
| NL | Gebruikershandleiding | 67 |
| PL | Podręcznik użytkownika | 80 |
| PT | Guia do utilizador | 93 |

Contents

| | |
|---|----|
| Contents | 2 |
| 1 Description and Intended Purpose | 3 |
| 2 Safety Information | 4 |
| 3 Construction | 5 |
| 4 Key Points and Precautions for Use | 6 |
| 5 Battery Charging | 7 |
| 5.1 Battery Level Indication | 9 |
| 5.2 Device Status Indication | 9 |
| Appendix 1 Quick Guide to LED Indications | 13 |

1 Description and Intended Purpose

These instructions are for the user. Please keep these instructions.

The term *device* is used throughout these instructions for use to refer to ElanIC.

Please read all these instructions for use, paying particular attention to the sections regarding safety and maintenance.

Application

The device is a biomimetic foot and ankle system with microprocessor controlled dorsiflexion and plantarflexion. It is designed to adapt dynamically (via motorized valves) to provide assistance when walking on a variety of surfaces, when changing gait speeds and increased levels of safety and security when walking and standing, up to the maximum user weight of 125 kg (275 lbs).

Features

- Dynamic damping resistance aims to improve gait characteristics enabling safer walking
- Microprocessor control of resistance in real time
- The device Software adapts to individual user gait characteristics
- Standing Support mode
- Aims to reduce stress at the socket interface
- PC interface setup with **Bluetooth**® link and direct communications
- Power is supplied via an integral rechargeable battery pack
- Recharging through easy-to-use magnetically self-aligning inductive charger
- Waterproof according to the parameters described in this manual

This device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve postural sway, balance stability and inter-limb loading symmetry while easing high pressures at the socket interface.

The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs).

2 Safety Information

The caution symbol highlights safety information which must be followed carefully.

1. The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs). Contact your service provider for advice if the device is to be used with any other electronic medical device.

2. Any changes in performance of the ankle motion e.g. restricted movement, non-smooth motion should be immediately reported to your service provider.

3. Always use a hand rail when descending stairs and at any other time if available.

4. Any excessive changes in heel height after programming will adversely affect ankle function and should be immediately reported to your service provider for reprogramming and calibration.

5. After continuous use or charging the ankle casing may become hot to the touch.

6. Avoid strong magnetic fields, sources of electrical interference.

7. Avoid exposure to extreme heat and/or cold.

8. The device is sealed for water resistance. However the water resistance of the device is limited by environmental factors and exposure time as detailed in this manual.

9. The inductive battery charger kit is not waterproof. Do not charge in atmospheres containing liquids and/or powders.

10. The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

11. Assembly, programming and repair of the device must only be carried out by suitably qualified practitioners, certified by Blatchford.

12. The user must not adjust or tamper with the setup of the device.

13. The user should be advised to contact their clinician if their condition changes.

14. The device detects standing and walking on inclines, declines and walking at various speeds on normal ground. Non-identified movements such as treadmills and travelators may cause unexpected ankle behaviour.

15. Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

16. To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

17. Be aware of finger trap hazard at all times


See further warnings relating to the charging of the device in Section 5 Battery Charging/Warnings.

3 Construction

Principal parts:

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium incorporating batteries (NiMH), electric motors and PCB)
- Carrier Assembly (aluminum/St. Stl.)
- Heel & Toe Springs (e-Carbon)
- Spring Attachment Screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)
- Charging Surface & Charger Pod (thermoplastic)

Component Identification

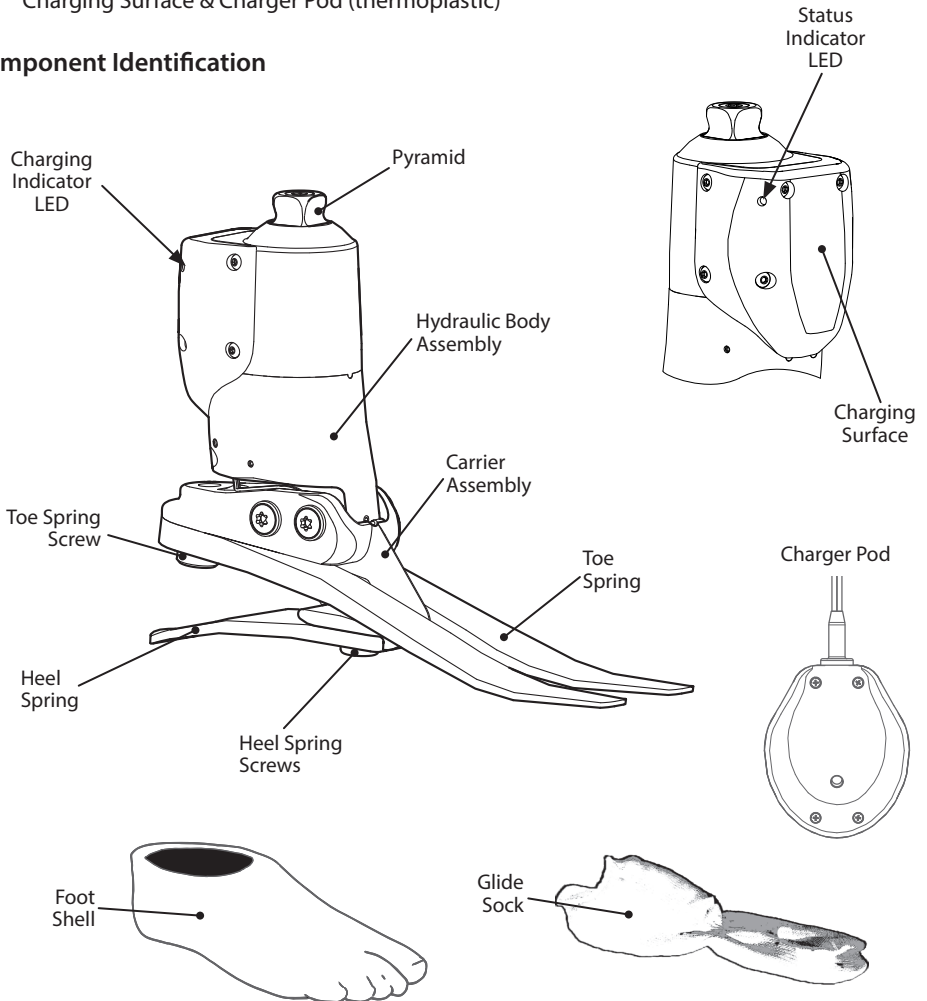4 Key Points and Precautions for Use

Maintenance must be carried out by competent personnel.

It is recommended that maintenance is carried out annually and a service carried out as a condition of warranty every 20 months.

A regular visual check of the foot is recommended, signs of wear or corrosion that may affect function should be reported to the service provider.

Key Points

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (freer movement)
- Any unusual noise (silent buzz from the device during charging is normal)
- Recurring rapid flashing of Charging Indicator/Status Indicator LEDs.

At low battery power levels the device will go into sleep mode, switch off dynamic control and use only the basic resistance settings and may affect **Bluetooth**® connectability. Charge the device batteries as soon as possible to restore Dynamic mode.

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Significant wear or excessive discoloration from long term exposure to UV.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, **DO NOT** use aggressive cleansers.

Transport and Storage

Storage temperature range between -15°C and 50°C (5 °F and 122 °F).

If the device is shipped, send the device and the charging pod in the packaging supplied with the unit. Always ship the device with the charger pod.

When built into a limb it should be stored securely so as not to damage the unit.

Lifting Loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is IP67 rated: waterproof to a maximum depth of 1 meter. Occasional submersion should be limited to a max. of 30 minutes. Dry after using in water.

Suitable for submersion

Avoid exposing the device to abrasive or corrosive environments, such as those containing sand, acids, salt, chlorinated, or chemically treated water e.g. swimming pools. Failure to comply will void the warranty. After any accidental exposure to such environments rinse the device in fresh water and dry thoroughly.

Exclusively for use between -15°C and 50°C (5°F and 122°F).

Charge only between 0 °C to 35 °C (32 °F to 95 °F)

5 Battery Charging

i Please familiarize yourself with the charging information in this section.
 The batteries are not field replaceable and will be replaced as necessary when serviced.

Read the user instructions supplied with the charger before using for the first time.



We recommend the device is charged daily; it should take 5.5 hours to charge fully, depending on residual charge. An exhausted or flat battery may take longer.

Setting Up

Lay the limb on a flat surface and connect the inductive charger pod to the charging surface on the back of the device.

Use the Charging Indicator LED on the right hand side of the device to confirm that charging has initiated correctly:

! Do not allow anything (e.g. stocking, clothing) to intrude in between the charging surface and the charger pod while charging is in progress. This will result in decreased charging performance and buildup of heat.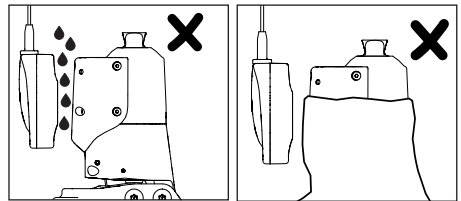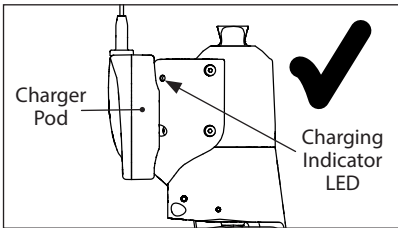

Charging

One initial flash x1 — Connection Established

Followed By

| LED | Charging state |
|--------------------------------|--------------------------------|
| Single Flash Repeated — — — | Charging - Battery Empty |
| Double Flash Repeated — — | Charging - Charge Level Medium |
| Triple Flash Repeated — | Charging - Charge Level High |
| Steady ON/Steady OFF / — | Charging Completed |
| Rapid Flashing | High Temperature Shut Down* |


*Charging continues when normal charging temperature is reached. (See Section 4.)

Reset

The device will take several seconds to reset when the charger is disconnected and a single blue flash from the Status Indicator LED on the left hand side of the device will confirm a correct startup.

Further information is provided by the charger pod LED (blue):

| LED | Error | Corrective Action |
|-------------------|--|--|
| Slowly Pulsating | No Error/Normal charging | Not needed |
| Steady Blue Light | Charger operational but not mated correctly to the charging surface of the device. | Try reattaching and check if anything prevents perfect mating of charging surfaces. Remove any intruding material. |
| No Light | The charger is not powered. | Check that charger is connected to the mains. |
| Rapid Pulsating | Possible overheating of the charger. | Charging may resume when normal charging temperature is reached. If recurring contact your service provider. |

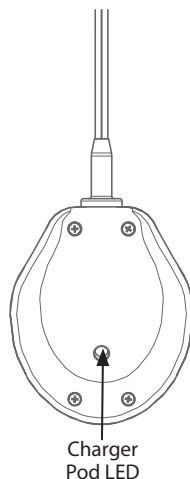Charging Error

Charging error is always indicated by a non-responding Charging Indicator LED on the right hand side of the device (no initial flash or charging state indication).

Warnings

- ⚠ 1. **The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs). Contact your service provider for advice if the device is to be used with any other electronic medical device.**
- ⚠ 2. **Only charge the batteries in the device using the charger supplied with the product. Do not use this charger for charging any other device.**
- ⚠ 3. **The battery charger can only be used with an electrical outlet that is rated between 100 V and 240 V AC, 50/60 Hz.**
- ⚠ 4. **Never connect the charger to the device while still wearing the limb.**
- ⚠ 5. **During charging no power is delivered to the ankle control system.**
- ⚠ 6. **The ankle and the charger may become warm during charging but should never be too hot to touch. There should be no abnormal smell. If this is the case immediately unplug from electrical supply and contact your service provider.**
- ⚠ 7. **Do not refit the limb until the charger has been disconnected.**
- ⚠ 8. **Always recharge the device before use if it has not been used for an extended period of time.**
- ⚠ 9. **The inductive battery charger kit is not waterproof. Do not charge in atmospheres containing liquids and/or powders.**
- ⚠ 10. **Do not exceed the charging temperature limitations set out in this manual.**
- ⚠ 11. **When charging an empty battery the charger may terminate prematurely (after a few minutes). If this happens, reconnect the charger. If this problem recurs contact your service provider.**
- ⚠ 12. **Do not leave the charger plugged to electrical supply after use.**


5.1 Battery Level Indication

To check the battery charge level, tilt the foot down vertically and hold stationary for 4 seconds. The Status Indicator LED on the left hand side of the device will flash blue, 1-3 times (see table) and then repeat to indicate the charge status. Tilt the foot again to repeat the battery status check.

Note: This feature can be switched off.

Note: After charging the device fully the Status Indicator LED may indicate 2 or 3 flashes. This will depend on the charging time, the age and condition of the battery.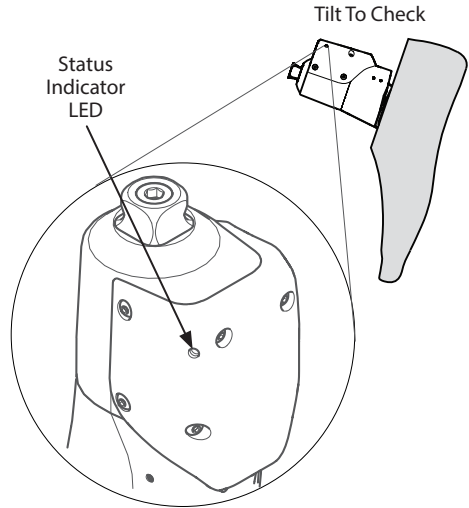

| LED | Battery Charge |
|-----------|----------------|
| 3 flashes | Maximum |
| 2 flashes | High |
| 1 flash | Low |

5.2 Device Status Indication

The same LED on the left hand side of the device is also used to indicate device status:

1. Single Flash
When the charger is disconnected it will flash once to indicate correct start up.
2. Steady ON
If after the charger is disconnected it stays on it indicates that an error was detected by your device.
3. Flashing rapidly
If it flashes rapidly, it indicates that **Bluetooth**® is active. This capability is to be used only by the clinician. The flashing can be ignored and will cease after a few minutes. Avoid holding the ankle horizontally for prolonged periods of time to prevent activating **Bluetooth**®.

See Appendix 1 for a full summary of all possible LED indications of the device and the charger pod.

FCC Compliance Statement

This device complies with part 15 of the FCC Rules.

Operation is subject to the following three conditions:-

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

Note... Contains pre-approved **Bluetooth**® module: FCC ID:T9J-RN41-1 IC: 6514A-RN413.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution: Exposure to Radio Frequency Radiation. This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party: Blatchford Inc.

1031 Byers Road, Miamisburg, OH 45342
800-548-3534 (toll free) | 937-291-3636
customerservice@blatchfordus.com

RSS Compliance Statement

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

1. this device may not cause interference, and
2. this device must accept any interference, including interference that may cause undesired operation of this device.

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Caution: Exposure to Radio Frequency Radiation.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website Blatchford Inc. 1031 Byers Road, Miamisburg, OH 45342.

800-548-3534 (toll free) | 937-291-3636 customerservice@blatchfordus.com

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical device

Single Patient – multiple use

Warranty

The device is warranted for – 36 months – foot shell 12 months - glide sock 3 months.

See Blatchford website for the current full warranty statement.

Scheduled service every 20 months is required, for maintenance of the warranty.

For extended warranty contact service provider.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following three conditions:-

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.
3. Contains pre-approved **Bluetooth**[®] module FCC ID: T9J-RN41-1

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

To help prevent potential harm to the environment or to human health from uncontrolled waste disposal, Blatchford offers a take-back service. Please contact Customer Services for details.

Trademark Acknowledgements

Bluetooth® is the registered trademark of **Bluetooth**® SIG.
Elan and Blatchford are registered trademarks of Blatchford Products Limited.

Manufacturer's Registered Address



Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Appendix 1 Quick Guide to LED Indications

Device Status Indicator LED (Left Hand Side)


Note: Take into consideration that the battery level indication feature may be disabled.

Note: The built-in **Bluetooth**® device can be activated only when the ankle is in a fully dorsiflexed position.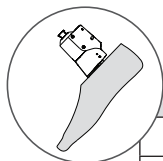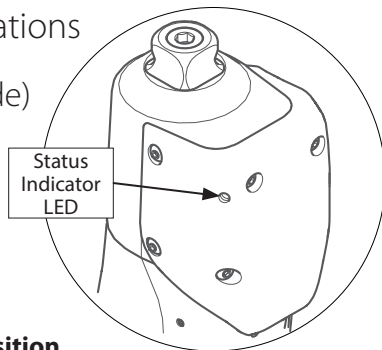

Tilted Position

| LED | Indicating | Action |
|---|--|--|
| Single Flash x2 | Battery level low | Charge to use |
| Double Flash x2 | Battery level high | Charge for maximum performance |
| Triple Flash x2 | Battery level high | Ready for use |
| Steady ON after battery level indication (foot dorsiflexed) | Bluetooth ® is ready for turning ON | Turn upright to turn Bluetooth ® ON |
| Steady ON without battery level indication (foot dorsiflexed) | Bluetooth ® is ready for turning ON (Battery level indication Disabled) | Turn upright to turn Bluetooth ® ON |

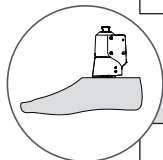Upright Position


| LED | Indicating | Action |
|---|---|---|
| Single flash (After disconnecting charger) | Correct start up after charging | None |
| Solid (After disconnecting charger) | Error detected | Contact service provider |
| Continuous flashing after disconnecting charger | Bluetooth ® is ready for connection | Wait few minutes for deactivation |
| Continuous flashing after Bluetooth ® turned On | Bluetooth ® is ready for connection | Connect PC software or Wait for deactivation |
| Steady ON after Bluetooth ® connection initiated | Bluetooth ® connection to PC software successful | Use PC software |
| Continuous flashing after Bluetooth ® connection established | Bluetooth ® connection is broken Bluetooth ® is ready for connection | Try reestablish connection Try reducing distance between devices |

Charging Indicator LED (Right Hand Side)

Note: Always interpret the indications given by the Charging Indicator LED and the Charger Pod LED in connection with each other.

Note: The charger will turn itself off after 6 hours of operation. This will manifest itself by the Charger Pod LED switching from pulsating to steady ON and the device Charging Indicator LED switching OFF.

This is normal and usually means that the device charged correctly. As soon as the charger turns itself off, the device immediately starts running on battery power (in sleep mode). If the device is left for extensive periods the battery will deplete.

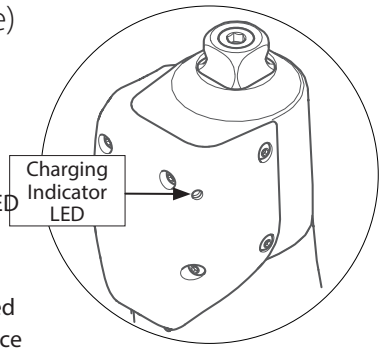| LED | Indicating | Action |
|------------------------------|--|--------------------------------|
| Start Up | | |
| OFF | Charger connection failed | Try reconnect |
| Primary Indications | | |
| Single flash | Correct start up after charger connected | See further indications below |
| Single flash + LED stays off | Correct start up after charger connected | Battery full |
| Single flash repeated | Charging empty battery | None |
| Double flash repeated | Charging medium level battery | None |
| Triple flash repeated | Charging high level battery | None |
| Rapid flashing | High temperature shut down | Charging resumes after cooling |
| End of Charging | | |
| Steady ON | Charging completed | Ready for use |

Charger Pod LED

| LED | Indicating | Action |
|-----------------|-----------------------------|--|
| Slow pulsating | Normal Operation of charger | None |
| Steady ON | Operational but no charging | Check mating to charging surface |
| No light | Not powered | Check mains connection |
| Rapid pulsating | Possible overheating | Charging may resume after cooling down |

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© Blatchford Products Limited 2021. All rights reserved.
Endolite India Ltd.
Blatchford Europe GmbH