Avalon^{K2}

User Guide

AV22L to AV30R
AV22LD to AV30RD

EN	User Guide	3
FR	Guide de l'Utilisateur	4
DE	Bedienungsanleitung für Anwender	5
IT	Guida per il Paziente	6
ES	Guia de Usuario	7
NO	Brukerhåndbok	8
RU	Руководство пользователя	9
TR	Kullanım Kılavuzu	10
AR	دليل المستخدم	11

Introduction

EN

This device is recommended for users that have the potential to achieve Activity Level 2 who may benefit from enhanced stability and an increase in confidence on uneven surfaces and slopes (maximum user weight 150kg).

The AvalonK2 dorsi flexes after mid stance and remains dorsi flexed at toe off through the swing phase to give greater toe clearance for improved confidence and security. The controlled plantar flexion provided by the AvalonK2 at heel contact improves time to foot flat which enhances stability on uneven surfaces and slopes. Please keep these instructions.

Key points for the user

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise

Any changes in body weight and/or activity level of the user must be reported to the clinical practitioner.

The AvalonK2 is water proof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15°C and 50°C.

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Avalon and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd.