Echelon^{VAC}

User Guide

EVAC22L1S—EVAC30R8S
EVAC22L1SD—EVAC30R8SD

EN	User Guide	1
FR	Guide de l'Utilisateur	3
DE	Bedienungsanleitung für Anwender	5
IT	Guida per il Paziente	7
ES	Guía de Usuario	9
NO	Brukerhåndbok	11
RU	Руководство пользователя	13
TR	Kullanım Klavuzu	15

EVAC22L1S–EVAC30R8S**Please keep these instructions.**

The EchelonVAC provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. In addition to the visco-elastic, self aligning hydraulic ankle it generates a vacuum in the range 12-17in Hg. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface. A moderate energy return foot with resilient multi-axial ankle movement. Independent heel and toe spring provides some axial deflection and the split toe provides good ground compliance.

-  **After continuous use the ankle casing may become hot to the touch.**
-  **To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.**
-  **Be aware of finger trap hazard at all times.**

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise
- Lack of vacuum

Note: Check that the suspension sleeve maintains a good seal with the skin proximally and socket distally.

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Discoloration of the residual limb.
- Significant wear or excessive discoloration from long term exposure to UV.

Intended for single user.

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers. Maintenance must be carried out by competent personnel. It is recommended that maintenance is carried out annually.

Environment:

Avoid exposing the EchelonVAC to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example, as these may promote premature wear.

Suitable for outdoor use

Exclusively for use between -15°C and 50°C (5°F to 122°F).

It is recommended that only Blatchford products be used in conjunction with the EchelonVAC.

Warranty

The EchelonVAC is warranted for 36 months – foot shell 12 months - glide sock 3 months.



This warranty does not apply to:

Consumable parts including the vacuum tubing, in-line filter and valves unless a failure has occurred due to a defect in materials or workmanship.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See Blatchford website for the current full warranty statement.

Vacuum System

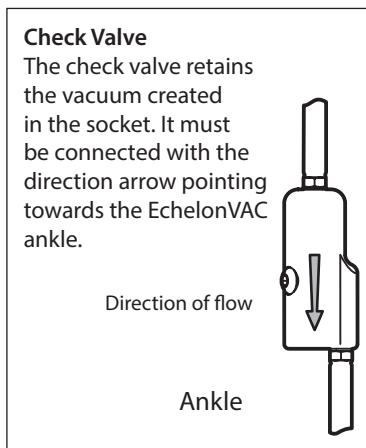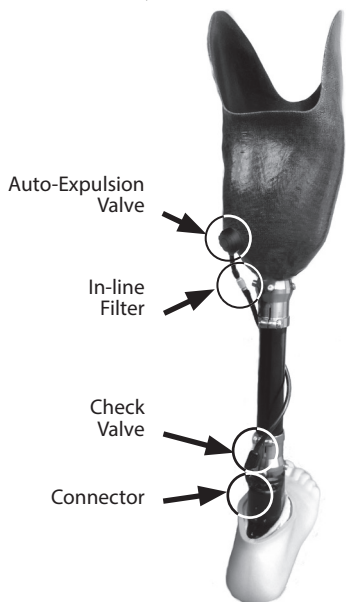Vacuum system guidance

	<i>Symptoms:</i>	<i>Cause/Remedy</i>
1.	Unable to generate a vacuum	Vacuum tube(s) disconnected Inspect and reconnect as necessary
		Limited ankle movement due to footwear a full range of movement is required
2.	Unable to maintain a vacuum	Vacuum tube(s) disconnected Inspect and reconnect as necessary
		Leakage at socket valve/barbs Consult your practitioner
		Check the integrity of the vacuum seal at the socket/residuum interface

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

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© Blatchford Products Limited 2019. All rights reserved.
Endolite India Ltd.