

# Silcare Liner Rectification Assistant

Patient Name \_\_\_\_\_ Reference Number \_\_\_\_\_ LT  RT


## Our recommended volume reduction is:

- 3-5% proximal reduction, dependent on soft tissue coverage and bony anatomy.
- 0-1% distal reduction (as above).
- Ensure your measurements are accurate to ensure effective total contact.

Volume reduction should mainly be carried out in the posterior soft tissue area. Minor reductions are permitted in the para-tibial areas to accentuate the bony anatomy.

## Sizes Available

22, 23.5, 25, 26.5, 28, 30, 32, 34, 36, 40

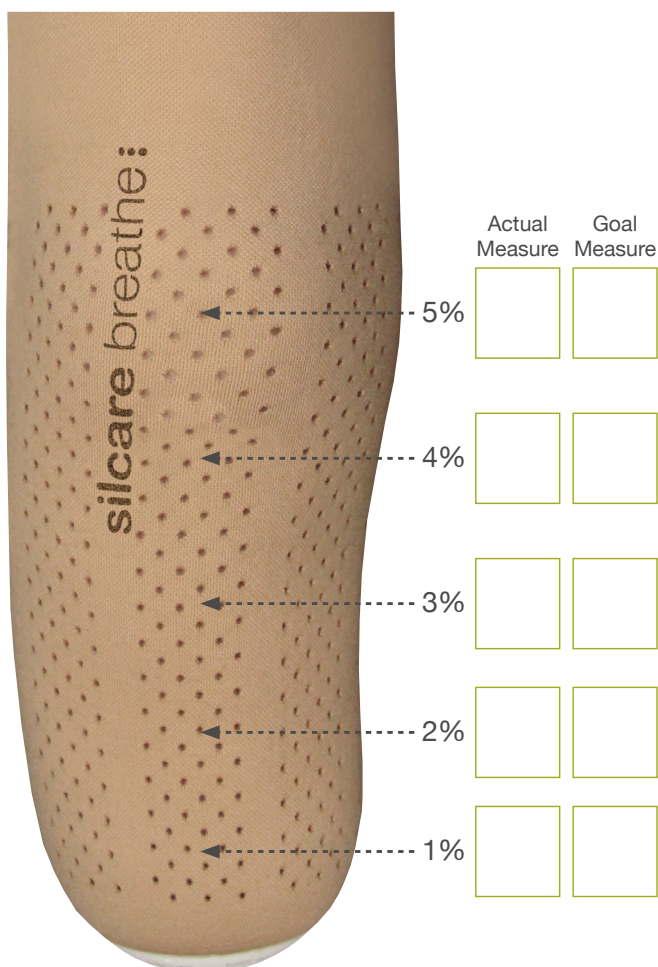

Measure (cm)	5% Reduction	4% Reduction	3% Reduction	2% Reduction	1% Reduction
55	52.25	52.8	53.35	53.9	54.45
54	51.3	51.84	52.38	52.92	53.46
53	50.35	50.88	51.41	51.94	52.47
52	49.4	49.92	50.44	50.96	51.48
51	48.45	48.96	49.47	49.98	50.49
50	47.5	48	48.5	49	49.5
49	46.55	47.04	47.53	48.02	48.51
48	45.6	46.08	46.56	47.04	47.52
47	44.65	45.12	45.59	46.06	46.53
46	43.7	44.16	44.62	45.08	45.54
45	42.75	43.2	43.65	44.1	44.55
44	41.8	42.24	42.68	43.12	43.56
43	40.85	41.28	41.71	42.14	42.57
42	39.9	40.32	40.74	41.16	41.58
41	38.95	39.36	39.77	40.18	40.59
40	38	38.4	38.8	39.2	39.6
39	37.05	37.44	37.83	38.22	38.61
38	36.1	36.48	36.86	37.24	37.62
37	35.15	35.52	35.89	36.26	36.63
36	34.2	34.56	34.92	35.28	35.64
35	33.25	33.6	33.95	34.3	34.65
34	32.3	32.64	32.98	33.32	33.66
33	31.35	31.68	32.01	32.34	32.67
32	30.4	30.72	31.04	31.36	31.68
31	29.45	29.76	30.07	30.38	30.69
30	28.5	28.8	29.1	29.4	29.7
29	27.55	27.84	28.13	28.42	28.71
28	26.6	26.88	27.16	27.44	27.72
27	25.65	25.92	26.19	26.46	26.73
26	24.7	24.96	25.22	25.48	25.74
25	23.75	24	24.25	24.5	24.75
24	22.8	23.04	23.28	23.52	23.76
23	21.85	22.08	22.31	22.54	22.77
22	20.9	21.12	21.34	21.56	21.78

Follow Blatchford: | blatchford.co.uk  
 @BlatchfordGrp | +44 (0) 1256 316600  
 /BlatchfordGrp | customer.service@blatchford.co.uk



blatchford