Silcare Breathe Cushion Liner

User Guide

SBTTCP22L - SBTTCP40L

| | | |
|----|----------------------------------|----|
| EN | User Guide | 2 |
| FR | Guide de l'Utilisateur | 4 |
| DE | Bedienungsanleitung für Anwender | 6 |
| IT | Guida per il Paziente | 8 |
| ES | Guia de Usuario | 10 |
| NO | Brukerhåndbok | 12 |

Blatchford:

Silcare Breathe SBTTCP22L - SBTTCP40L

Please keep these instructions.

The Silcare Breathe silicone liner provides a perforated cushioned socket interface which allows moisture to pass through the perforations keeping the skin dry. It is manufactured from biocompatible materials.

 **Wearers with sensitive skin, diabetics and vascular cases should be extra vigilant and may need to apply lubricant to sensitive areas. We recommend a routine visual check and if required consult your medical Practitioner.**

 Take care when handling the liner to avoid possible contamination with materials such as fibreglass which will stick to the liner and cause skin irritation.

Donning and doffing


 Care must be taken during donning/doffing not to damage the liner with fingernails or sharp jewelry.

 Do **NOT** pull or stretch the liner.

1. **Roll** the liner inside out.
2. Make sure the liner is positioned correctly to suit the top profile.
3. **Roll** the liner on taking care to expel any trapped air.

 Warning: when donning a sock, clothing and the prosthetic limb, be aware that the liner can build up a static charge

The fabric cover may ladder slightly around the perforations. This will not affect the function or effectiveness of the liner.

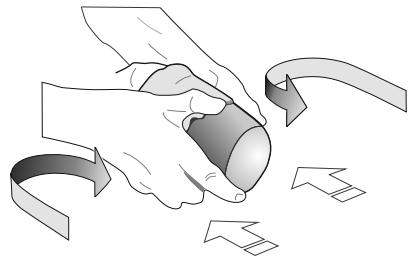Cleaning and care

Wash the inside of the liner daily to avoid any build up of bacteria. Use a simple, unperfumed soap (ideally pH balanced).

Hand wash or Cool machine wash.

Perforated liners should be filled with warm water, the proximal (open end) held closed and the liner gently squeezed to clear /flush the perforations.

Rinse well in clean water to remove all residues and dry thoroughly before use.

Take care when drying and handling the liner. Pat dry with a lint-free cloth or leave to air dry. Do **NOT** leave the liner inside out to dry as this can stretch the liner and permanently distort the shape.

Take care handling the liner when it is inside out to avoid picking up dust, grit and other contamination which could cause irritation to the skin.

Skin care

It is recommended that when using the liner the residuum should be washed daily with a simple, unperfumed soap (ideally pH balanced). Rinse thoroughly to remove any soapy residue before drying thoroughly. Use a lotion for dry skin. Make sure that any damaged skin is properly and suitably dressed.

Do **NOT** tumble dry.

Do **NOT** use alcohol sprays, household cleaners or abrasives which could damage the liner and cause skin irritation.

Note:

Perspiration may discolour some socket materials.

Precautions

Warning: To avoid danger of suffocation, keep the liner away from babies and children.

Keep away from direct heat sources which could damage the liner.

Stop using the liner if it is torn or damaged.

Stop using the liner if perforations become enlarged as this can cause skin to become trapped and blistered.

For other medical conditions please check with your physician or medical practitioner for following this recommendation for skin care.

Any deterioration in the residuum condition should be reported to your practitioner.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Silcare Breathe and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd